


QUICK GUIDE

AVALO VCCC Resuscitation Cart

A.

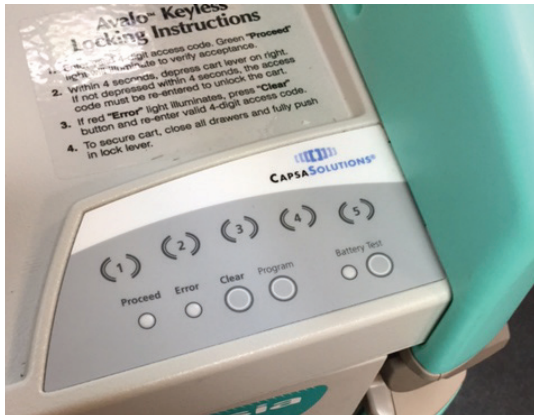

Opening Drawers

1. Enter 1234 (unless changed) on keypad
2. Wait for unlocking sound
3. Pull lever down and out

B.



Locking Drawers

Push lever in and up

C.



Locking the Wheels

RED= Engage red foot pedal for brake
BLACK= Directional castor

D.



Adjustable Drawer Dividers

E.



Manual Override

1. Insert override key into battery panel (rear of cart)
2. Push while turning
3. Simultaneously turn lever on front of cart down and out

Paragon Care Group Pty Ltd
11 Dalmore Drive
Scoresby 3173 VIC Australia

T 1300 369 559
F 03 8833 7890
E info@paragoncare.com.au

paragoncare.com.au